

Bead Block®



Embolic beads for the embolisation of arteriovenous malformations, benign prostatic hyperplasia*, and hypervascular tumours including uterine fibroids*

DESCRIPTION

Bead Block® is a preformed, deformable blue-tinted microsphere consisting of a biocompatible, acrylamido polyvinyl alcohol macromer. Bead Block® comprises a range of hydrogel microspheres that are biocompatible, hydrophilic, nonresorbable and precisely calibrated.

INDICATIONS

Bead Block® is intended to be used for the embolisation of hypervascular tumours, including uterine fibroids* and arteriovenous malformations (AVMs). Bead Block® is also intended to be used for the treatment of symptomatic benign prostatic hyperplasia (BPH).*

PRESENTATION

- Single-syringe pack
 - Each pack contains a single 20ml syringe presented in a sterile, sealed pre-formed Tyvek® peel-away tray
 - Each syringe contains approximately 2ml of Bead Block® in physiological buffered saline (total volume approximately 6ml)
 - Each syringe is intended for single patient use
- Two-year shelf life from manufacture
- Steam sterilised

* Size restrictions apply to this indication. Please see table below.

| PRODUCT CODES | | | INDICATION | | |
|------------------------|------------------|--------------|---|-------------------------------------|--|
| Bead Block® Size Range | BTG Product Code | Label Colour | Hypervascular Tumours and Arteriovenous malformations | Uterine Fibroid Embolisation (UFE)* | Prostatic Artery Embolisation (PAE) for BPH* |
| Bead Block® 100-300µm | EB2S103 | Yellow | ✓ Yes | ✗ No | ✓ Yes |
| Bead Block® 300-500µm | EB2S305 | Blue | ✓ Yes | ✗ No | ✓ Yes |
| Bead Block® 500-700µm | EB2S507 | Red | ✓ Yes | ✓ Yes | ✗ No |
| Bead Block® 700-900µm | EB2S709 | Green | ✓ Yes | ✓ Yes | ✗ No |
| Bead Block® 900-1200µm | EB2S912 | Purple | ✓ Yes | ✓ Yes | ✗ No |

- Bead Block® are precisely calibrated for targeted embolisation
- Bead Block® are compressible for smooth passage through microcatheter. They regain their spherical shape quickly for effective vessel occlusion
- Bead Block® resist attrition, fragmentation and aggregation, all of which may compromise embolisation outcomes



Bead Block®

BTG CUSTOMER SERVICES TEAM: customer.services@btgplc.com

| | | | |
|---------|---------------------|-----------------|---------------------|
| AUSTRIA | tel. +43-720880739 | NETHERLANDS | tel. +31-202410932 |
| BELGIUM | tel. +32-28087916 | PORTUGAL | tel. +351-308802698 |
| FRANCE | tel. +33-184884056 | SPAIN | tel. +34-911981738 |
| GERMANY | tel. +49-3030809275 | SWITZERLAND | tel. +41-315280675 |
| IRELAND | tel. +353-15134139 | UK | tel. +44-2033187881 |
| ITALY | tel. +39-0694805067 | OTHER COUNTRIES | tel. +44-2033187881 |

MANUFACTURE AND CERTIFICATION

Bead Block® is manufactured by Biocompatibles UK Ltd, a BTG International group company. CE Mark 0086, Steam sterilised, single-use, non-pyrogenic, latex-free.

Biocompatibles UK Ltd,

a BTG International group company,
Chapman House, Farnham Business Park
Weydon Lane, Farnham
Surrey, GU9 8QL, UK

PATENTS

US 5,583,163
US 6,652,883
US 6,676,971
US 7,070,809

EXPLANATION OF SYMBOLS ON BEAD BLOCK® PACKAGE LABELS



Product fulfils conformity assessment according to Council Directive 93/42/EEC verified by BSi. Product class: IIb

REF

Catalogue number



Batch number / Lot number



Do not reuse



Attention see instructions for use



Steam sterilised



Use before / Expiry



Protect from light



Protect from moisture



Do not freeze

Imagine where we can go.



BTG